
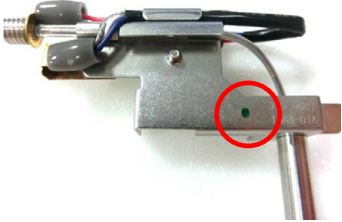




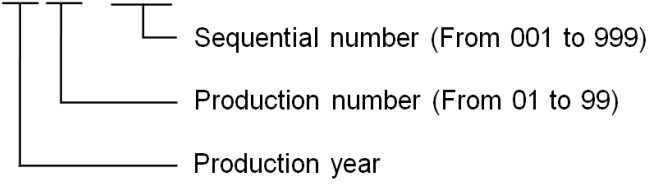

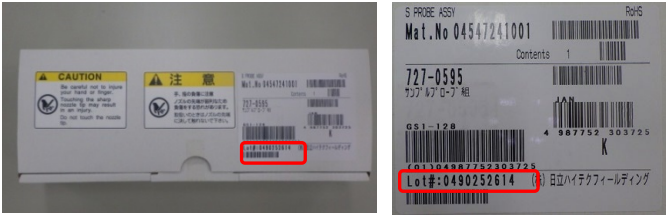
**Target**

With this procedure potentially affected Sample Probes can be identified both already installed Sample Probes and still boxed Sample Probes.

**Identification of potentially affected sample probes for cobas c501/ cobas c502 and cobas c311**


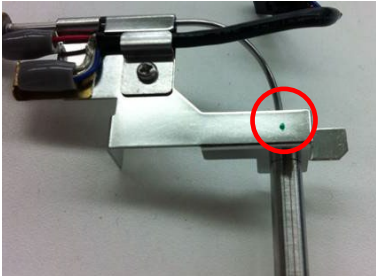



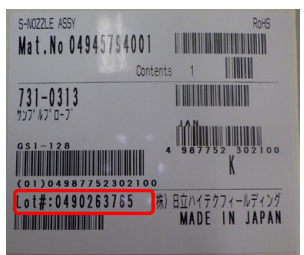
There are two HHT production plants NAKA & OMUTA producing Sample Probes. Potentially affected Sample Probes can be easily encountered by the following procedure. If a yellow dot is applied to the Sample Probe connector the Sample Probe is already equipped with a modified connector.

04547241001 PROBE SAMPLE (cobas c501/ cobas c502)	
05899427001 PROBE SAMPLE S (cobas c311)	
Affected serial number which printed on the probe	<p>NAKA; 1509-001 onward, and <u>no marked yellow dot</u> on the connector.                  OMUTA; 1535-001 onward, and <u>no marked yellow dot</u> on the connector.</p> <p>Refer to Place of yellow dot on the connector in this table for detail.</p>
Identification of production site in HHT	<div style="display: flex; justify-content: space-around;">   </div> <p>Either <span style="border: 1px solid black; padding: 2px;">NAKA</span> or <span style="border: 1px solid black; padding: 2px;">OMUTA</span> is indicated on the original package.                  Either Black or Green dot is marked on the sample probe.</p> <p>Black dot: Produced by <span style="border: 1px solid black; padding: 2px;">NAKA</span>                  Green dot: Produced by <span style="border: 1px solid black; padding: 2px;">OMUTA</span></p>
Printed place on sample probe	<div style="display: flex; justify-content: space-around;">   </div> <p>Refer to Rule of printed serial number for detail.</p>

<p>Rule of printed serial number</p>	<p>727-0587 ← Part number                  YY AA – BBB ← Serial number</p>  <p>Sequential number (From 001 to 999)                  Production number (From 01 to 99)                  Production year</p> <p>Serial number printed on the probes for <b>cobas</b> c501/c502 and <b>cobas</b> c311 are making a common numbering by HHT's two production lines.</p>
<p>Place of yellow dot on the connector</p>	 <p><u>No yellow dot mark</u> on the connector is affected.                  If the yellow mark on the connector is existed, the probe was modified done by HHT despite if the Serial number which printed on the probe was met.</p>
<p>SAP lot number which printed on the original package</p>	
<p>Affected Sample Probe for <b>cobas</b> c311 (P/N 724-0920)</p>	<p>Lot number which indicated on the logistics label                  0490232027, 0490233855, 0490236427, 0490238543, 0490241937, 0490248322, 0490250880, 0490235713, 0490252029, 0490256963</p>
<p>Affected Sample Probe for <b>cobas</b> c 501 and c502 (P/N 727-0595)</p>	<p>0490233762, 0490233764, 0490233766, 0490233767, 0490233777, 0490235904, 0490235905, 0490235906, 0490235907, 0490235949, 0490235950, 0490237594, 0490237595, 0490237353, 0490237354, 0490237597, 0490237598, 0490237600, 0490237602, 0490237604, 0490237606, 0490238955, 0490238957, 0490238893, 0490238895, 0490238896, 0490238898, 0490238899, 0490242149, 0490242151, 0490242153, 0490242155, 0490242327, 0490242328, 0490242329, 0490242330, 0490242331, 0490250491, 0490250492, 0490250493, 0490250494, 0490250495, 0490250496, 0490251032, 0490251033, 0490251034, 0490251036, 0490252524, 0490252527, 0490252529, 0490252532, 0490252247, 0490252249, 0490252251, 0490252252, 0490252272, 0490252398, 0490254550, 0490254551, 0490254552, 0490254553, 0490254554, 0490254555, 0490254804, 0490254805, 0490254806, 0490254807, 0490256999, 0490257000, 0490256993, 0490256994, 0490256995, 0490256996, 0490256997, 0490256998, 0490254808, 0490257001, 0490257847, 0490257848, 0490257849, 0490257850, 0490257851, 0490258652</p>

**Identification of potentially affected sample probes for cobas c701/ cobas c702**

HHT production plants NAKA & OMUTA producing Sample Probes as well for **cobas c701/ cobas c702** and ISE. Potentially affected Sample Probes can be encountered by the following procedure.

04945794001 SAMPLE PROBE ( <b>cobas c701/ cobas c702</b> )	
Affected serial number which printed on the probe	ISE and Sample Probe for <b>cobas c701/ cobas c702</b> do not have serial number printed on the probe itself.
Identification of production site in HHT	  <p>Either <b>NAKA</b> or <b>OMUTA</b> is indicated on the original package.                  Either Black or Green dot is marked on the sample probe.</p> <p>Black dot : Produced by <b>NAKA</b>                  Green dot : Produced by <b>OMUTA</b></p>
Place of yellow dot on the connector	  <p><u>No yellow dot mark on the connector is affected.</u>                  If the yellow mark on the connector is existed, the Sample Probe was already modified by HHT.</p>
SAP lot number which printed on the original package	  <p>Refer to the separate attachment for detail of SAP lot numbers which printed on the original package.</p>

Affected Sample Probe for <b>cobas</b> c701/ <b>cobas</b> c702 (P/N 731-0313)	0490238960, 0490238961, 0490238961, 0490238961, 0490238963, 0490242332, 0490251040, 0490251041, 0490252546, 0490252547, 0490252547, 0490254810, 0490254811, 0490221658, 0490223228, 0490223229, 0490225255, 0490226172, 0490226173, 0490226174, 0490227626, 0490227627, 0490227628, 0490230276, 0490230277, 0490230279, 0490232203, 0490232204, 0490232205, 0490232206, 0490233778, 0490233779, 0490233780, 0490233781, 0490233783, 0490235909, 0490235910, 0490235911, 0490235912, 0490235913, 0490237365, 0490237366, 0490237367, 0490237368, 0490237369, 0490237370, 0490238901, 0490238902, 0490238903, 0490238904, 0490242160, 0490242161, 0490242163, 0490242165, 0490242166, 0490250498, 0490250499, 0490250500, 0490250501, 0490250502, 0490252308, 0490252312, 0490252318, 0490252320, 0490252322, 0490254558, 0490254559, 0490254560, 0490254561, 0490254562, 0490257035, 0490257036, 0490257037, 0490257038, 0490257039, 0490257854, 0490257855, 0490257856, 0490257857, 0490257858
---	--

<end of the document>