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Closing Confirmation Sheet

DRI OCT Triton, DRI OCT Triton plus Software, Ver.1.1.4

Customer name	Installation date / /		
	Agency/Supplier	Business office	
Address			
TEL () -		The body model:	The body S/N:

● Version Up check procedure of a software

	Procedure item	Check point	PASS	FALL																	
1	It be ver.1.1.3 in front of Version Up. (Select "information" from the "SETUP" menu.) <table border="1" style="margin-left: 20px;"> <tr> <td>Basic</td> <td>Serial No</td> <td></td> </tr> <tr> <td></td> <td>Region Code</td> <td></td> </tr> <tr> <td></td> <td>System</td> <td>1.1.3</td> </tr> <tr> <td></td> <td>OCT Capture Count</td> <td></td> </tr> <tr> <td></td> <td>Fundus Capture Count</td> <td></td> </tr> </table>	Basic	Serial No			Region Code			System	1.1.3		OCT Capture Count			Fundus Capture Count		It's confirmed that it's a repair target.	<input type="checkbox"/>	<input type="checkbox"/>		
Basic	Serial No																				
	Region Code																				
	System	1.1.3																			
	OCT Capture Count																				
	Fundus Capture Count																				
2	Version Up, error confirmation	An error message don't be indicated.	<input type="checkbox"/>	<input type="checkbox"/>																	
3	Version behind Version Up be Ver.1.1.4. (Select "information" from the "SETUP" menu.) <table border="1" style="margin-left: 20px;"> <tr> <td>Basic</td> <td>Serial No</td> <td></td> </tr> <tr> <td></td> <td>Region Code</td> <td></td> </tr> <tr> <td></td> <td>System</td> <td>1.1.4</td> </tr> <tr> <td></td> <td>OCT Capture Count</td> <td></td> </tr> <tr> <td></td> <td>Fundus Capture Count</td> <td></td> </tr> </table>	Basic	Serial No			Region Code			System	1.1.4		OCT Capture Count			Fundus Capture Count		Make sure that the version information is as follows. <table border="1" style="margin-left: 20px;"> <tr> <td>System</td> <td>1.1.4</td> </tr> </table>	System	1.1.4	<input type="checkbox"/>	<input type="checkbox"/>
Basic	Serial No																				
	Region Code																				
	System	1.1.4																			
	OCT Capture Count																				
	Fundus Capture Count																				
System	1.1.4																				
4	Checking existing data	Check existing data by the tool software "PowerSaveLRChecker". If wrong data exist make necessary correction. <table border="1" style="margin-left: 20px;"> <tr> <td>Nuber of checked images</td> <td></td> </tr> <tr> <td>Hours taken for the check</td> <td></td> </tr> </table>	Nuber of checked images		Hours taken for the check		<input type="checkbox"/>	<input type="checkbox"/>													
Nuber of checked images																					
Hours taken for the check																					

Comment

Name	Signature	Initials	Date
TESTING PERFORMED BY			
CUSTOMER			