

---

To our customers with  
Dräger Atlan workstations

August 2025

## **Important safety notice!** Possible failure of the automatic ventilation due to impurities in the motor electronics

**This only affects Atlan anaesthesia workstations with serial numbers  
according to the attached list.  
Other anaesthesia machines are NOT affected!**

Dear Sir or Madam,

In the course of our global market surveillance, we have become aware of cases in which the Atlan anaesthesia workstations either indicated a failure of the piston ventilator before use or suffered a failure of the mechanical ventilation during use. There have not been any reports of patient injuries to date.

Our investigations revealed that the cases are due to impurities (dust) that were introduced for a certain time frame during the manufacturing process of the ventilator motor encoder. According to our records, you have received one or more devices from the affected serial number range.

**If the error occurs before use, e.g. in standby or system test mode, mechanical ventilation cannot be started. If the error occurs during a procedure, the mechanical ventilation may fail, and the device alerts the user by displaying the "Ventilator failure!!!" message. Manual ventilation or spontaneous breathing remains possible in both cases. It may be necessary to manually ventilate the patient to prevent patient injury. With ongoing alarm of the ventilation failure the Atlan continues to provide the Man/Spon ventilation mode, fresh-gas and agent delivery as well as all monitoring functions without restriction.**

[Redacted]

[Redacted]

[Redacted]

[Redacted]

Our Dräger Service will contact you and arrange an appointment for replacing the motor, free of charge.

**Until then, please continue operating your devices with the customary care by only using the device under permanent supervision in line with the Atlan instructions for use. If the mechanical ventilation fails, use Man/Spon ventilation mode to ventilate the patient manually as needed.**

If necessary, you can downgrade the alarm priority of the "Ventilator failure!!!" alarm with "ALARM RESET".

We regret any inconveniences this may cause.

Please ensure that all users of the Dräger Atlan as well as other persons concerned within your organization are made aware of this Important Safety Notice. Please complete and return the attached reply card to confirm this.

If you have made the products available to third parties, please forward a copy of this Important Safety Notice to them.

Please store this Important Safety Notice at least until the motor has been replaced.

The responsible authorities have been notified of this action.

**Identification of affected medical devices:**

According to our records, you have received at least one Atlan anaesthesia workstation manufactured by Drägerwerk AG & Co. KGaA (European Single Registration Number: DE-MF-000005329) that may be affected by this issue.

**Contact:**

If you have any questions, please do not hesitate to contact your local Dräger representative. We apologise for any inconvenience this measure may cause.

Yours faithfully,



### Attachment to Important Safety Notice – list of affected devices

Device	Part number	Basic UDI DI
Atlan A100	8621100	040486751301035FK19X030W4
Atlan A100 XL	8621200	
Atlan A300	8621300	040486751301035FK19Z000W9
Atlan A300 XL	8621400	
Atlan A350	8621500	040486751301035FK19T010V2
Atlan A350 XL	8621600	

#### UDIs for Atlan A100

(01)4048675594963(11)250204(21)ASUA-0333(93)8621100-08	(01)4048675594963(11)250404(21)ASUC-0335(93)8621100-09
(01)4048675594963(11)250204(21)ASUA-0334(93)8621100-08	(01)4048675594963(11)250404(21)ASUC-0336(93)8621100-09
(01)4048675594963(11)250204(21)ASUA-0336(93)8621100-08	(01)4048675594963(11)250411(21)ASUD-0045(93)8621100-09
(01)4048675594963(11)250210(21)ASUA-0457(93)8621100-08	(01)4048675594963(11)250422(21)ASUD-0148(93)8621100-09
(01)4048675594963(11)250220(21)ASUB-0166(93)8621100-08	(01)4048675594963(11)250425(21)ASUD-0218(93)8621100-09
(01)4048675594963(11)250320(21)ASUC-0002(93)8621100-09	(01)4048675594963(11)250425(21)ASUD-0229(93)8621100-09
(01)4048675594963(11)250320(21)ASUC-0003(93)8621100-09	(01)4048675594963(11)250425(21)ASUD-0230(93)8621100-09
(01)4048675594963(11)250401(21)ASUC-0260(93)8621100-09	(01)4048675594963(11)250425(21)ASUD-0231(93)8621100-09
(01)4048675594963(11)250401(21)ASUC-0261(93)8621100-09	(01)4048675594963(11)250428(21)ASUD-0240(93)8621100-09
(01)4048675594963(11)250402(21)ASUC-0280(93)8621100-09	(01)4048675594963(11)250428(21)ASUD-0241(93)8621100-09
(01)4048675594963(11)250402(21)ASUC-0281(93)8621100-09	(01)4048675594963(11)250428(21)ASUD-0242(93)8621100-09
(01)4048675594963(11)250402(21)ASUC-0282(93)8621100-09	(01)4048675594963(11)250428(21)ASUD-0243(93)8621100-09
(01)4048675594963(11)250404(21)ASUC-0330(93)8621100-09	(01)4048675594963(11)250429(21)ASUD-0255(93)8621100-09
(01)4048675594963(11)250404(21)ASUC-0333(93)8621100-09	(01)4048675594963(11)250429(21)ASUD-0256(93)8621100-09
(01)4048675594963(11)250404(21)ASUC-0334(93)8621100-09	(01)4048675594963(11)250429(21)ASUD-0257(93)8621100-09

#### UDIs for Atlan A100 XL

(01)4048675594956(11)250214(21)ASUB-0079(93)8621200-08	(01)4048675594956(11)250401(21)ASUC-0258(93)8621200-09
(01)4048675594956(11)250214(21)ASUB-0080(93)8621200-08	(01)4048675594956(11)250401(21)ASUC-0259(93)8621200-09
(01)4048675594956(11)250214(21)ASUB-0084(93)8621200-08	(01)4048675594956(11)250417(21)ASUD-0139(93)8621200-09
(01)4048675594956(11)250214(21)ASUB-0085(93)8621200-08	(01)4048675594956(11)250422(21)ASUD-0161(93)8621200-09
(01)4048675594956(11)250214(21)ASUB-0086(93)8621200-08	(01)4048675594956(11)250422(21)ASUD-0162(93)8621200-09
(01)4048675594956(11)250214(21)ASUB-0087(93)8621200-08	(01)4048675594956(11)250422(21)ASUD-0163(93)8621200-09
(01)4048675594956(11)250225(21)ASUB-0212(93)8621200-08	

#### UDIs for Atlan A300

(01)04048675556343(11)250207(21)ASUA-0431(93)8621300-19	(01)04048675556343(11)250407(21)ASUC-0374(93)8621300-20
(01)04048675556343(11)250210(21)ASUA-0462(93)8621300-19	(01)04048675556343(11)250407(21)ASUC-0375(93)8621300-20
(01)04048675556343(11)250210(21)ASUA-0463(93)8621300-19	(01)04048675556343(11)250408(21)ASUC-0411(93)8621300-20
(01)04048675556343(11)250210(21)ASUA-0464(93)8621300-19	(01)04048675556343(11)250408(21)ASUC-0412(93)8621300-20
(01)04048675556343(11)250210(21)ASUA-0467(93)8621300-19	(01)04048675556343(11)250408(21)ASUC-0413(93)8621300-20
(01)04048675556343(11)250211(21)ASUA-0494(93)8621300-19	(01)04048675556343(11)250408(21)ASUC-0414(93)8621300-20
(01)04048675556343(11)250212(21)ASUB-0019(93)8621300-19	(01)04048675556343(11)250408(21)ASUC-0415(93)8621300-20
(01)04048675556343(11)250212(21)ASUB-0049(93)8621300-19	(01)04048675556343(11)250410(21)ASUD-0040(93)8621300-20
(01)04048675556343(11)250213(21)ASUB-0050(93)8621300-19	(01)04048675556343(11)250417(21)ASUD-0124(93)8621300-20
(01)04048675556343(11)250213(21)ASUB-0051(93)8621300-19	(01)04048675556343(11)250417(21)ASUD-0125(93)8621300-20
(01)04048675556343(11)250213(21)ASUB-0055(93)8621300-19	(01)04048675556343(11)250417(21)ASUD-0132(93)8621300-20
(01)04048675556343(11)250213(21)ASUB-0056(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0151(93)8621300-20
(01)04048675556343(11)250214(21)ASUB-0057(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0152(93)8621300-20
(01)04048675556343(11)250214(21)ASUB-0058(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0153(93)8621300-20
(01)04048675556343(11)250214(21)ASUB-0059(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0154(93)8621300-20
(01)04048675556343(11)250214(21)ASUB-0071(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0155(93)8621300-20
(01)04048675556343(11)250214(21)ASUB-0072(93)8621300-19	(01)04048675556343(11)250422(21)ASUD-0156(93)8621300-20
(01)04048675556343(11)250217(21)ASUB-0089(93)8621300-19	(01)04048675556343(11)250423(21)ASUD-0169(93)8621300-20
(01)04048675556343(11)250217(21)ASUB-0090(93)8621300-19	(01)04048675556343(11)250423(21)ASUD-0170(93)8621300-20
(01)04048675556343(11)250217(21)ASUB-0091(93)8621300-19	(01)04048675556343(11)250423(21)ASUD-0171(93)8621300-20















(01)4048675556183(11)250326(21)ASUC-0175(93)8621600-20 | (01)4048675556183(11)250502(21)ASUD-0294(93)8621600-20

### Ventilator kit assembly

Manufacturing month	Serial numbers	
11-2024	ASTM-...	...-1468, ...-1469, ...-1470, ...-1473, ...-1474, ...-1475, ...-1476, ...-1477, ...-1478, ...-1479, ...-1480, ...-1481, ...-1482, ...-1485, ...-1488, ...-1490, ...-1491, ...-1492, ...-1493, ...-1494, ...-1495, ...-1498, ...-1501, ...-1502, ...-1503, ...-1508, ...-1509, ...-1510
01-2025	ASUA-...	...-0384, ...-0389, ...-0538, ...-0539, ...-0540, ...-0541, ...-0542, ...-0543, ...-0544, ...-0545, ...-0546, ...-0547, ...-0548, ...-0549, ...-0550, ...-0551, ...-0552, ...-0554, ...-0555, ...-0557, ...-0562, ...-0584, ...-0595, ...-0596, ...-0597, ...-0598, ...-0599, ...-0600, ...-0601, ...-0602, ...-0603, ...-0604, ...-0605, ...-0606, ...-0607, ...-0608, ...-0609, ...-0610, ...-0611, ...-0613, ...-0614, ...-0627, ...-0629, ...-0630, ...-0631, ...-0632, ...-0633, ...-0638, ...-0640, ...-0641, ...-0642, ...-0646, ...-0649, ...-0653, ...-0694, ...-0708, ...-0712, ...-0713, ...-0714, ...-0715, ...-0716, ...-0717, ...-0718, ...-0719, ...-0720, ...-0721, ...-0722, ...-0723, ...-0724, ...-0725, ...-0726, ...-0727, ...-0728, ...-0729, ...-0730, ...-0731, ...-0732, ...-0733, ...-0734, ...-0735, ...-0736, ...-0737, ...-0738, ...-0739, ...-0740, ...-0741, ...-0742, ...-0743, ...-0744, ...-0745, ...-0746, ...-0747, ...-0748, ...-0749, ...-0750, ...-0751, ...-0752, ...-0753, ...-0754, ...-0755, ...-0756, ...-0757, ...-0759, ...-0761
02-2025	ASUB-...	...-0013, ...-0014, ...-0015, ...-0016, ...-0017, ...-0019, ...-0020, ...-0021, ...-0022, ...-0023, ...-0024, ...-0025, ...-0026, ...-0028, ...-0030, ...-0031, ...-0032, ...-0033, ...-0034, ...-0037, ...-0038, ...-0039, ...-0040, ...-0042, ...-0043, ...-0044, ...-0045, ...-0046, ...-0048, ...-0049, ...-0051, ...-0052, ...-0053, ...-0054, ...-0055, ...-0056, ...-0058, ...-0059, ...-0060, ...-0061, ...-0062, ...-0063, ...-0065, ...-0066, ...-0068, ...-0069, ...-0071, ...-0077, ...-0080, ...-0081, ...-0082, ...-0087, ...-0088, ...-0089, ...-0090, ...-0094, ...-0095, ...-0096, ...-0097, ...-0098, ...-0103, ...-0106, ...-0107, ...-0108, ...-0109, ...-0110, ...-0111, ...-0112, ...-0114, ...-0117, ...-0118, ...-0119, ...-0120, ...-0121, ...-0123, ...-0124, ...-0125, ...-0126, ...-0128, ...-0129, ...-0130, ...-0132, ...-0133, ...-0135, ...-0136, ...-0138, ...-0141, ...-0142, ...-0144, ...-0149, ...-0150, ...-0152, ...-0154, ...-0156, ...-0157, ...-0158, ...-0159, ...-0160, ...-0161
03-2025	ASUC-...	...-0246, ...-0247, ...-0248, ...-0249, ...-0250, ...-0251, ...-0252, ...-0253, ...-0254, ...-0255, ...-0256, ...-0257, ...-0258, ...-0259, ...-0260, ...-0261, ...-0262, ...-0263, ...-0264, ...-0265, ...-0266, ...-0267, ...-0268, ...-0269, ...-0270, ...-0271, ...-0272, ...-0273, ...-0274, ...-0275, ...-0276, ...-0277, ...-0278, ...-0279, ...-0280, ...-0281, ...-0282, ...-0283, ...-0284, ...-0285, ...-0286, ...-0287, ...-0288, ...-0289, ...-0290, ...-0291, ...-0292, ...-0293, ...-0294, ...-0295, ...-0296, ...-0297, ...-0298, ...-0299, ...-0300, ...-0301, ...-0302, ...-0303, ...-0304, ...-0305, ...-0306, ...-0307, ...-0308, ...-0309, ...-0310, ...-0311, ...-0312, ...-0313, ...-0314, ...-0315, ...-0316, ...-0317, ...-0318, ...-0319, ...-0320, ...-0321, ...-0322, ...-0323, ...-0324, ...-0325, ...-0326, ...-0327, ...-0328, ...-0329, ...-0330,

		...-0331, ...-0332, ...-0333, ...-0334, ...-0335, ...-0336, ...-0337, ...-0338, ...-0339, ...-0340, ...-0341,
04-2025	ASUD-...	...-0001, ...-0002, ...-0004, ...-0005, ...-0007, ...-0008, ...-0011, ...-0012, ...-0013, ...-0014, ...-0016, ...-0017, ...-0020, ...-0021, ...-0023, ...-0024, ...-0025, ...-0026, ...-0027, ...-0028, ...-0029, ...-0030, ...-0031, ...-0032, ...-0033, ...-0034, ...-0035, ...-0036, ...-0037, ...-0038, ...-0039, ...-0040, ...-0041, ...-0042, ...-0043, ...-0044, ...-0045, ...-0046, ...-0047, ...-0048, ...-0049, ...-0050, ...-0051, ...-0052, ...-0053, ...-0054, ...-0055, ...-0056, ...-0057, ...-0058, ...-0059, ...-0060, ...-0061, ...-0062, ...-0063, ...-0064, ...-0065, ...-0066, ...-0067, ...-0068, ...-0069, ...-0070, ...-0071, ...-0072, ...-0073, ...-0074, ...-0075, ...-0076, ...-0077, ...-0078, ...-0079, ...-0080, ...-0081, ...-0082, ...-0084, ...-0085, ...-0086, ...-0087, ...-0088, ...-0089, ...-0090, ...-0091, ...-0092, ...-0093, ...-0094, ...-0095, ...-0096, ...-0097, ...-0098, ...-0099, ...-0100

## Reply Card

**Please fax or e-mail this form to your Dräger representative!**

D R Ä G E R	To: < To be completed by relevant subsidiaries/dealers >
	Department: _____
	Dräger Representative _____
	Fax: _____
	Phone: _____
	E-mail: _____
<b>Re: Important Safety Notice on Dräger Atlan, August 2025</b>	
(Please complete)	
C U S T O M E R	Hospital: _____
	Customer name: _____
	Phone: _____ Fax: _____
	E-mail: _____
	Address: _____
	Address 2: _____
D A T A	Country: _____
	Town/city: _____
	<b>Quantity of affected units:</b> _____
<input type="checkbox"/> We acknowledge receipt of the safety notice and that the information contained therein has been brought to the attention of all affected users.	
(Please complete and sign)	
Title/position: _____	
Name: _____	(Please print in capitals)
Signature: _____	Date: _____