



FSN Ref: 2025-11(02)
Date : 10 Dec.2025.

FSCA Ref: 2025-11(02)

Urgent Field Safety Notice
Mölnlycke® Procedure Trays

For Attention of: Theatre Manager

Contact details of local representative (name, e-mail, telephone, address etc.)
Name: Local Customer Care contact will be added for each specific market
Email: XXX.XXX@molnlycke.com
Telephone: +XXXXXXXXXXXXXXXXXX



FSN Ref: 2025-11(02)
Date : 10 Dec.2025.

FSCA Ref: 2025-11(02)

Urgent Field Safety Notice (FSN)
Mednet component - Syringe 10ml L/L 3parts Red plunger within
Mölnlycke® Procedure Trays

1. Information on Affected Devices					
1.	<p style="text-align: center;">1. Device Type(s)</p> <table border="1" style="width: 100%;"> <tr> <td style="width: 50%;">Component Name:</td> <td>Mölnlycke Component code</td> </tr> <tr> <td>Syringe 10ml L/L 3parts Red plunger</td> <td>2313673-00</td> </tr> </table> <p>Included in various Mölnlycke® Procedure Trays Mölnlycke® Procedure Trays consist of customized configurations of components, which are assembled and delivered sterile within one packaging solution.</p>	Component Name:	Mölnlycke Component code	Syringe 10ml L/L 3parts Red plunger	2313673-00
Component Name:	Mölnlycke Component code				
Syringe 10ml L/L 3parts Red plunger	2313673-00				
1.	<p style="text-align: center;">2. Commercial name(s)</p> <p>See Appendix I Product Table</p>				
1.	<p style="text-align: center;">3. Primary clinical purpose of device(s)</p> <p>Color Coded Syringes are intended to inject fluids into or withdraw fluids from patients.</p> <p>The clinical purpose of Mölnlycke® Procedure Trays is to provide a customized sterile co-packing of components for different clinical interventions.</p>				
1.	<p style="text-align: center;">4. Device Model/Catalogue/part number(s)</p> <p>See Appendix I Product Table</p>				
1.	<p style="text-align: center;">5. Affected serial or lot number range</p> <p>See Appendix I Product Table</p>				

2 Reason for Field Safety Corrective Action (FSCA)	
2	<p style="text-align: center;">1. Description of the product problem*</p> <p>Mölnlycke has recently been informed by MedNet GmbH, the legal manufacturer of the device in question, that they are issuing an urgent field safety notice after contamination with particles was identified in some syringes on a specific batch, which Mölnlycke includes in some of the Mölnlycke® Procedure trays.</p> <p>MedNet GmbH investigations in the production environment have revealed that 90% of the specified batches are not affected. For the remaining 10%, contamination cannot be definitively ruled out. Therefore, the aforementioned batches are being recalled in their entirety.</p> <p>Mölnlycke has decided to follow the legal manufacturer FSN and do a Recall of all potentially affected trays where this component is included.</p>
2	<p style="text-align: center;">2. Hazard giving rise to the FSCA*</p> <p>Information from MedNet Field Safety Notice:</p> <p>MedNet identified that these particles may not be detectable before use due to the potential for the particles to adhere to the syringe seal; therefore, in the worst-case scenario, their use on patients could lead to an embolism.</p>



FSN Ref: 2025-11(02)
Date : 10 Dec.2025.

FSCA Ref: 2025-11(02)

3. Type of Action to mitigate the risk	
3.	<p>1. Action To Be Taken by the User</p> <p><input checked="" type="checkbox"/> Identify Device <input checked="" type="checkbox"/> Return Device</p> <p>We need your help in ensuring that all affected products are located and that below actions are performed.</p> <p>Please follow below instructions:</p> <ol style="list-style-type: none"> 1. Identify and isolate the unused Mölnlycke® Procedure Trays at your facility, please see Appendix I for affected product information. 2. Fill out the Customer reply form or Distributor reply form with the quantity of identified affected products. Please sign the Customer reply form or Distributor reply form and send email to _____ per its instruction within 10 business days. 3. Even if you no longer have concerned Mölnlycke Trays, fill out the Customer Reply Form or Distributor Reply Form and return it back within 10 business days. Mölnlycke needs to be sure all customers are aware of the situation. 4. Mölnlycke will contact you and arrange for collection of the product(s) from your facility, as soon as you return the Reply Form. Mölnlycke will issue a credit for the goods returned. 5. If you have forwarded any affected products to other healthcare institutions, please send them a copy of this Field Safety Notice. Make sure they act accordingly. 6. If you are a distributor, please inform your customers by sending them a copy of this Field Safety Notice. Make sure they act accordingly. Please return the Customer reply form to you and return it to Mölnlycke. <p>In addition, Mölnlycke appreciates your help in collecting data on product complaints and/or incidents related to the product concerned. Please follow the reporting procedures established by your facility.</p>
3.	<p>7. Is customer Reply Required?</p> <p style="text-align: right;">Yes (Within 10 business days)</p>

4. General Information		
4.	1. FSN Type	New
4.	2. Further advice or information already expected in follow-up FSN?	No
4.	3. Manufacturer information (For contact details of local representative refer to page 1 of this FSN)	
	a. Company Name	Mölnlycke Health Care AB
	b. Address	
	c. Website address	
4.	4. The Competent (Regulatory) Authority of your country has been informed about this communication to customers.	
4.	5. List of attachments/appendices:	Appendix I Product table Customer Reply Form Distributor Reply Form
4.	6. Name/Signature	
	Electronically signed by: Reason: Date: Dec 10, 2025 13:34:48 GMT+1	

Transmission of this Field Safety Notice	
	<p>This notice needs to be passed on all those who need to be aware within your organisation or to any organisation where the potentially affected devices have been transferred. (As appropriate)</p> <p>Please transfer this notice to other organisations on which this action has an impact. (As appropriate)</p> <p>Please maintain awareness on this notice and resulting action for an appropriate period to ensure effectiveness of the corrective action.</p> <p>Please report all device-related incidents to the manufacturer, distributor or local representative, and the national Competent Authority if appropriate, as this provides important feedback.</p>

FSN Ref: 2025-11(02)
Date : 10 Dec.2025.

FSCA Ref: 2025-11(02)

Appendix I

To be added for each market

FSN Ref: 2025-11 (02)
Date: 10.Dec.2025.

FSCA Ref: 2025-11 (02)

Customer Reply Form

1. Field Safety Notice (FSN) information	
FSN Reference number	2025-11 (02)
FSN Date	10.Dec.2025.
Product/ Device name	See Appendix I Product table
Product Code(s)	See Appendix I Product table
Batch/Serial Number (s)	See Appendix I Product table

2. Customer Details	
Account Number	
Healthcare Organisation Name*	
Organisation Address*	
Department/Unit	
Shipping address if different to above	
Contact Name*	
Title or Function	
Telephone number*	
Email*	

3. Customer action undertaken on behalf of Healthcare Organisation																							
<input type="checkbox"/>	I confirm receipt of the Field Safety Notice and that I read and understood its content. I do not have any affected devices.																						
<input type="checkbox"/>	I confirm receipt of the Field Safety Notice and that I read and understood its content. I have quarantined the unused trays with affected component ready for return - enter number of devices ready for return	<table border="1"> <thead> <tr> <th>Quantity</th> <th>Article/Material Number</th> <th>Lot/Batch Number</th> </tr> </thead> <tbody> <tr><td> </td><td> </td><td> </td></tr> <tr><td> </td><td> </td><td> </td></tr> <tr><td> </td><td> </td><td> </td></tr> <tr><td> </td><td> </td><td> </td></tr> <tr><td> </td><td> </td><td> </td></tr> <tr> <td>N/A</td> <td colspan="2">Comments:</td> </tr> </tbody> </table>	Quantity	Article/Material Number	Lot/Batch Number																N/A	Comments:	
Quantity	Article/Material Number	Lot/Batch Number																					
N/A	Comments:																						
Print Name*																							
Signature*																							
Date*																							



FSN Ref: 2025-11 (02)

FSCA Ref: 2025-11 (02)

Date: 10.Dec.2025.

4. Return acknowledgement to sender	
Email	
Customer Helpline	
Postal Address	Mölnlycke Health Care AB,
Fax	
Deadline for returning the customer reply form*	Within 10 days

Mandatory fields are marked with *

It is important that your organisation takes the actions detailed in the FSN and confirms that you have received the FSN.

Your organisation's reply is the evidence we need to monitor the progress of the corrective actions.

FSN Ref: 2025-11(02)
Date: 10 Dec 2025

FSCA Ref: 2025-11(02)

Distributor Reply Form

1. Field Safety Notice (FSN) information	
FSN Reference number*	2025-11(02)
FSN Date*	10 Dec 2025
Product/ Device name*	See Appendix I Product table
Product Code(s)	See Appendix I Product table
Batch/Serial Number (s)	See Appendix I Product table

2. Distributor Details	
Company Name*	
Account Number	
Address*	
Shipping address if different from above	
Contact Name*	
Title or Function	
Telephone number*	
Email*	

3. Return acknowledgement to Sender	
Email	
Distributor Helpline	
Postal Address	
Web Portal	
Deadline for returning the Distributor reply form*	Within 10 business days

4. Distributors (Tick all that apply)			
<input type="checkbox"/>	*I confirm the receipt, the reading and understanding of the Field Safety Notice.		
<input type="checkbox"/>	I have checked my stock and quarantined affected trays		
<input type="checkbox"/>	I have identified customers that received or may have received this affected trays		
<input type="checkbox"/>	I have attached customer list		
<input type="checkbox"/>	I have informed the identified customers of this FSN	Date of communication:	
<input type="checkbox"/>	I have received confirmation of reply from all identified customers		
<input type="checkbox"/>	I have quarantined affected trays ready for return – enter the number of quarantined devices ready for return	Quantity	Article/Material Number
			Lot/Batch Number

FSN Ref: 2025-11(02)

FSCA Ref: 2025-11(02)

Date: 10.Dec.2025

		N/A	Comments:	
<input type="checkbox"/>	Neither I nor any of my customers has any affected trays in inventory			
Print Name*				
Signature*				
Date *				

Mandatory fields are marked with *

It is important that your organisation takes the actions detailed in the FSN and confirms that you have received the FSN.

Your organisation's reply is the evidence we need to monitor the progress of the corrective actions.

Table of Affected Products



Mölnlycke ref: 2025-11 (02) – Mölnlycke® Procedure Trays

Country	Product	LOT	Product Name	GMDN code	GMDN term
AT	97098819-10	25037606	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25059999	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25050076	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25050077	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25090595	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25090658	Radialis PPak Klinikum Wels	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
DE	97127606-01	25021349	TAVI Set	P16545	Angiography kit
		25021351	TAVI Set	P16545	Angiography kit
		25094295	TAVI Set	P16545	Angiography kit
		25187123	TAVI Set	P16545	Angiography kit
		25187124	TAVI Set	P16545	Angiography kit
		25187125	TAVI Set	P16545	Angiography kit
	97130159-00	25153490	Angio Set Radiologie Koblenz	P16545	Angiography kit
		25173854	Angio Set Radiologie Koblenz	P16545	Angiography kit
		25114577	Angio Set Radiologie Koblenz	P16545	Angiography kit
		25114578	Angio Set Radiologie Koblenz	P16545	Angiography kit
ES	97049383-04	25043703	Pack de Angiografia Coronaria	P16545	Angiography kit
		25099699	Pack de Angiografia Coronaria	P16545	Angiography kit
		25119901	Pack de Angiografia Coronaria	P16545	Angiography kit
	97290180-51	25028753	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25036411	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25052004	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25062832	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25062858	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25062859	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25062860	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25063785	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25063810	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25064979	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25066620	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25070586	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25073966	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25073991	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25073992	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25088045	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25088195	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25089233	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25090952	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25091440	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit

Country	Product	LOT	Product Name	GMDN code	GMDN term
		25114978	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25115074	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25115075	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25115076	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25115077	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25116235	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25116349	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25118614	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25119709	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25151958	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25152043	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25152044	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25152045	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25152131	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25153987	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25154040	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25156145	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25156238	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25219175	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25219176	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25219177	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25219180	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25229772	Pack de Angiografia Femoral y Radial 971	P16545	Angiography kit
		25017148	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057480	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057498	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057551	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057554	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057556	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25057557	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073746	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073851	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073853	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073855	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073857	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25073859	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25100950	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25101013	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25101016	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25101018	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25101020	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25101022	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25102900	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25103024	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25103027	PERUSANGIOGRAFIA SETTI HUS KARDIO	P16545	Angiography kit
		25039413	Cardio-vasculaire hybride	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25085179	Cardio-vasculaire hybride	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25090519	Cardio-vasculaire hybride	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
FI	97001961-14				
FR	97112529-01				

Country	Product	LOT	Product Name	GMDN code	GMDN term
		25090643	Cardio-vasculaire hybride	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25090644	Cardio-vasculaire hybride	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
	97138812-00	25097495	Lot 1 Angiographie N°2 (4 abords & 1 cas	P16545	Angiography kit
IT	97086574-04	25035781	SET CORONAROGRAFIA	P16545	Angiography kit
		25122127	SET CORONAROGRAFIA	P16545	Angiography kit
		25122248	SET CORONAROGRAFIA	P16545	Angiography kit
	97086575-02	25063408	SET PM	P16545	Angiography kit
		25144305	SET PM	P16545	Angiography kit
		25161980	SET PM	P16545	Angiography kit
	97086785-02	25042580	ENDOVASCOLARE	P16545	Angiography kit
		25058656	ENDOVASCOLARE	P16545	Angiography kit
		25091713	ENDOVASCOLARE	P16545	Angiography kit
		25103002	ENDOVASCOLARE	P16545	Angiography kit
		25162448	ENDOVASCOLARE	P16545	Angiography kit
	97138233-00	25021802	SET EMODINAMICA AQ	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
	97138233-01	25103308	EMODINAMICA AQ	P16545	Angiography kit
		25103309	EMODINAMICA AQ	P16545	Angiography kit
		25103310	EMODINAMICA AQ	P16545	Angiography kit
		25103311	EMODINAMICA AQ	P16545	Angiography kit
		25113504	EMODINAMICA AQ	P16545	Angiography kit
		25113505	EMODINAMICA AQ	P16545	Angiography kit
		25113506	EMODINAMICA AQ	P16545	Angiography kit
		25160144	EMODINAMICA AQ	P16545	Angiography kit
25160145	EMODINAMICA AQ	P16545	Angiography kit		
NL	97042438-09	25051100	Cardiopack ADRZ	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25094035	Cardiopack ADRZ	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
		25094134	Cardiopack ADRZ	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
	97065390-23	25215751	Cath Pack	P16545	Angiography kit
		25215752	Cath Pack	P16545	Angiography kit
		25216619	Cath Pack	P16545	Angiography kit
		25216620	Cath Pack	P16545	Angiography kit
PL	97107430-10	25044710	zestaw do zabiegów wewnątrznaczyniowych	P16545	Angiography kit
		25044721	zestaw do zabiegów wewnątrznaczyniowych	P16545	Angiography kit
	97120006-01	25062943	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25079149	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25070635	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25114024	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25132930	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25174087	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25174155	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25192898	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
		25213268	Zestaw do angiografii udowo - promieniow	P16545	Angiography kit
97120317-02	25051158	Zestaw do zabiegów hybrydowych USK Zielo	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use	

Country	Product	LOT	Product Name	GMDN code	GMDN term
		25087082	Zestaw do zabiegów hybrydowych USK Zielo	P61371	Cardiothoracic surgical procedure kit, non-medicated, single-use
	97121907-01	25084710	Zestaw do angiografii naczyń mózgowych D	P16545	Angiography kit
	97126311-02	25021337	ANGIOGRAFIA	P16545	Angiography kit
		25037212	ANGIOGRAFIA	P16545	Angiography kit
		25037214	ANGIOGRAFIA	P16545	Angiography kit
		25065017	ANGIOGRAFIA	P16545	Angiography kit
		25065048	ANGIOGRAFIA	P16545	Angiography kit
		25065050	ANGIOGRAFIA	P16545	Angiography kit
		25065072	ANGIOGRAFIA	P16545	Angiography kit
		25065074	ANGIOGRAFIA	P16545	Angiography kit
		25065077	ANGIOGRAFIA	P16545	Angiography kit
	97144111-01	25088994	PO_GCM_Angiografia	P16545	Angiography kit
		25089036	PO_GCM_Angiografia	P16545	Angiography kit
		25089038	PO_GCM_Angiografia	P16545	Angiography kit
		25089039	PO_GCM_Angiografia	P16545	Angiography kit
		25090781	PO_GCM_Angiografia	P16545	Angiography kit
		25091566	PO_GCM_Angiografia	P16545	Angiography kit
		25146033	PO_GCM_Angiografia	P16545	Angiography kit
		25146084	PO_GCM_Angiografia	P16545	Angiography kit
		25146087	PO_GCM_Angiografia	P16545	Angiography kit
		25146089	PO_GCM_Angiografia	P16545	Angiography kit
	25146090	PO_GCM_Angiografia	P16545	Angiography kit	
	97144111-02	25194141	PO_GCM_Angiografia	P16545	Angiography kit
		25194142	PO_GCM_Angiografia	P16545	Angiography kit
		25194143	PO_GCM_Angiografia	P16545	Angiography kit
		25194144	PO_GCM_Angiografia	P16545	Angiography kit
		25194145	PO_GCM_Angiografia	P16545	Angiography kit
		25194146	PO_GCM_Angiografia	P16545	Angiography kit
		25194147	PO_GCM_Angiografia	P16545	Angiography kit
		25194148	PO_GCM_Angiografia	P16545	Angiography kit
		25194149	PO_GCM_Angiografia	P16545	Angiography kit
		25194150	PO_GCM_Angiografia	P16545	Angiography kit
SA	97053171-13	25064781	Customized Efficiency Vascular Kit-KFSH	P16545	Angiography kit
		25096592	Customized Efficiency Vascular Kit-KFSH	P16545	Angiography kit
		25107442	Customized Efficiency Vascular Kit-KFSH	P16545	Angiography kit
SE	97050998-15	25020065	Angioset Uppsala	P16545	Angiography kit
		25084924	Angioset Uppsala	P16545	Angiography kit
		25122039	Angioset Uppsala	P16545	Angiography kit
		25138154	Angioset Uppsala	P16545	Angiography kit
		25151681	Angioset Uppsala	P16545	Angiography kit
		25153520	Angioset Uppsala	P16545	Angiography kit